



Just Read

The Simple – Easy to Use Video Magnifier

**Thank you for purchasing an *Optima Just Read*
video magnifier, we hope you enjoy using it.**

User Instructions



Please read the user instructions carefully and familiarise yourself with this Instruction leaflet before operating this device.



- Do not look at the sun or other bright light sources through the magnifying Screen. This may damage your eyes.
- Do not look directly at the LED light as they are very bright.
- Do not point the LED lights directly at others.
- Always handle the magnifier with care.
- Do not get the magnifier wet or operate it with wet hands.
- Keep the magnifier away from direct heat sources.


Warranty:

This video magnifier is guaranteed against all manufacturing defects either in material or workmanship for 1 year from the date of purchase.


Normal wear and tear, misuse or abuse (accidental or otherwise) such as impact by dropping or damage caused through exposure to heat or moisture is not covered under the warranty. If the magnifier is defective, please return it to the place of purchase with proof of purchase.

Getting to know your *Just Read*


TOP VIEW

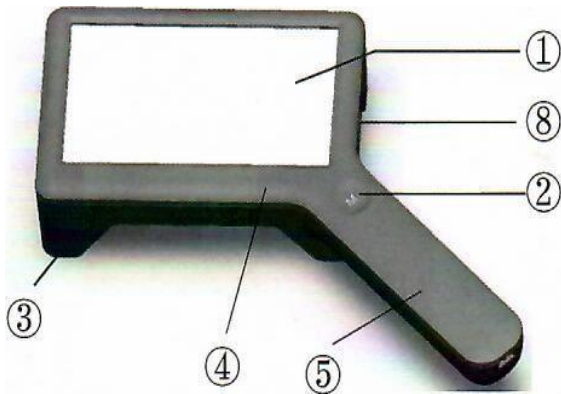

UNDERNEATH VIEW

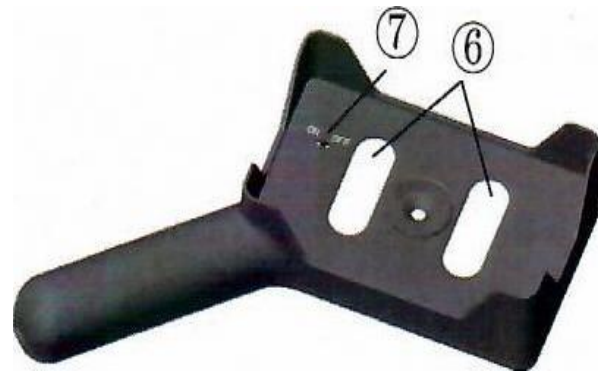

1	Screen
2	Magnification Change Button
3	Support Frame
4	Screen Frame
5	Integral Battery
6	LED Lights
7	Power Switch
8	USB Charge Point

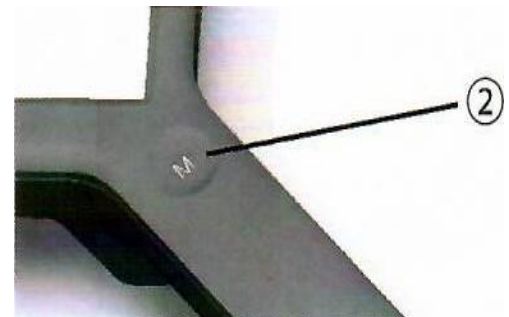

To Operate

- 1 Turn **ON** using the slide switch (7) on the underside.
- 2 Press the button (2) on the handle to switch on the screen.
- 3 Press button (2) again to change magnification.
- 4 There are 3 levels of magnification.
Press once = 3X magnification
Press twice = 5X magnification.
Press three = 10X magnification
Press four = switch off the screen.

Do not forget to slide the Power switch (7) Off when finished.

To charge the battery connect the USB connection to the port (8) on the side of the screen frame and the other end into a USB power supply.

Specifications

Lens Size	4.3" 91x56mm
Magnification	3X, 5X, 10X,
Illumination	6 White LEDs
Power	3.7v 1200mAh
Operating Time	Approx.2 Hours
Size	181x413x32mm
Weight	Approx. 145g.

For information about other Optima Low Vision products or services please visit:

www.optimalowvision.co.uk

Email:sales@optimalowvision.co.uk

Tel. 01803 864218

